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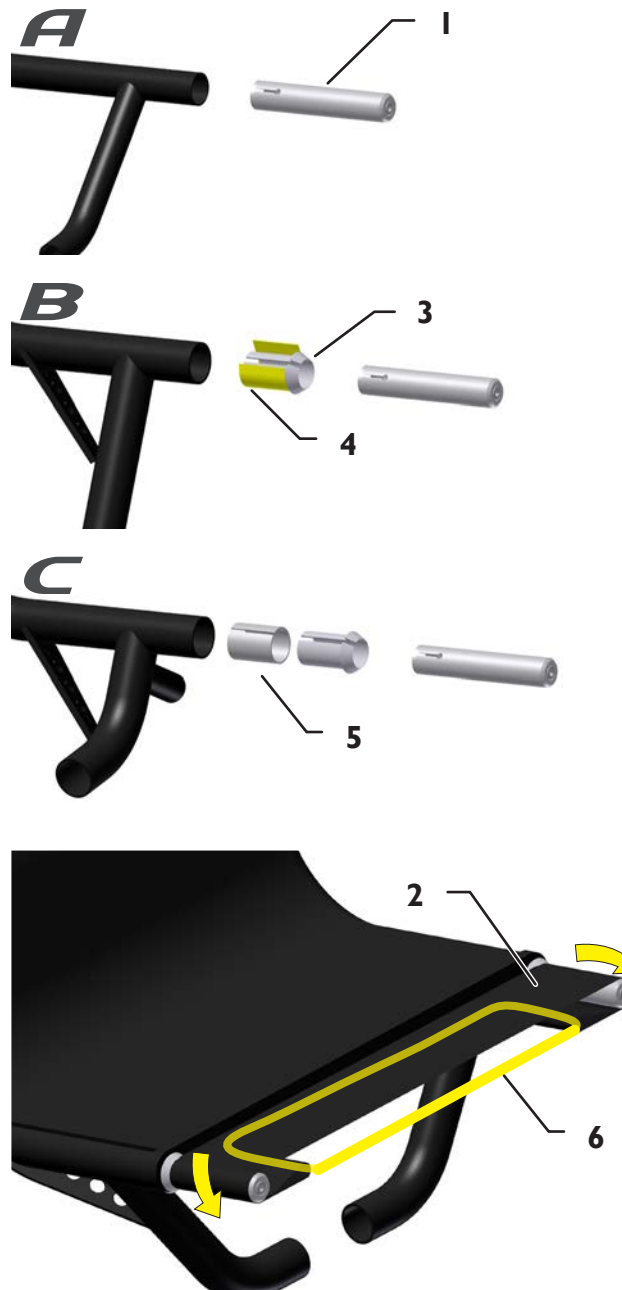
The Seat Extension increases the surface area of the seat or, alternatively, the seat back. This optional feature offers you more space and comfort and is available for the Kettwiesel, Lepus, and Pino.

### Installation Instructions

1. Open the Velcro seam at the front of the seat and remove the plastic caps from the ends of the seat tubes.

#### Installation on the Lepus - Figure A

- 2a. Insert the seat extension clamps (1) approx. 4 cm (1½") into the ends of the seat tubes. The length of the seat extension clamps protruding from the ends of the seat tubes should be equivalent to the width of the strip of webbing. Tighten the bolts.

#### Installation on the Kettwiesel - Figure B

- 2b. Wrap the grip tape (4) around the XXL adapter sleeves (3), and insert the sleeves with tape into the ends of the seat tubes. Insert the seat extension clamps approx. 4 cm (1½") into the adapter sleeves. The length of the seat extension clamps protruding from the ends of the seat tubes should be equivalent to the width of the strip of webbing. Tighten the bolts.

#### Note:

When installing the seat extension clamps to Kettwiesel seats produced before mid-2014, the Pino adapter sleeves (5) must be used instead of the tape to fit the XXL adapter sleeves to the inner diameter of the seat tubes.

#### Installation on the Pino - Figure C

- 2c. Slide the Pino adapter sleeves (5) over the XXL adapter sleeves. Be sure that the side slits of the two adapter sleeves are lined up. Insert the sleeves into the seat tubes. Insert the seat extension clamps approx. 4 cm (1½") into the adapter sleeves. The length of the seat extension clamps protruding from the ends of the seat tubes should be equivalent to the width of the strip of webbing. Tighten the bolts.
3. Wrap the strip of webbing around the two seat extensions and pull the webbing taut from below using the tensioning straps (6).
4. Attach the Velcro seam of the seat cover to the strip of webbing.

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